GIRARDELLI X-3D

C20-0010
2201

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Automatic x-ray processor

- Instruction for use -

Ritter Concept GmbH
Freiburger Str. 45
88400 Biberach, Germany
Fon: +49 7351 52 925-10
Fax: +49 7351 52 925-11

e-mail: info@ritterconcept.com
Internet: <http://www.ritterconcept.com>

 **Ritter**[®]
[THE DENTAL EXPERTS]

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Automatic x-ray processor

Dimensions Abmessungen:	
Width:	60 cm (23.62")
Depth:	32 cm (12.60")
Height with upper part	33 cm (12.99")
Colour	
Upper part:	white
Lower part:	blue-grey
Weight:	≈7 kg
Rated voltage:	220-240 V, 50/60 Hz
Rated input:	250 W
Fuse:	2x M 2,5 A L, 250 V
Run-through time:	
Film clip with red transport shaft	≈7 min (drying time included)
Film clip with black transport shaft	≈8.5 min (drying time included)
Filmformats:	
all small formats	
2x3 cm or 3x4 cm	
Capacity of the baths	
Developer bath:	≈1.8 l
Fixer bath:	≈1.8 l
Water baths:	≈1.8 l
Operating conditions	
Temperature range:	0°C to +40°C (+32°F to +104°F)
rel. humidity:	20% to 85% (no condensation)
Transport and storage conditions	
Temperature range:	-20°C to +60°C (-4°F to +140°F)
rel. humidity:	5% to 90% (no condensation)
Valid standard:	DIN EN 60601-1

Technical modifications reserved!

In order to use the device properly, please read the instruction for use carefully before starting to operate. In addition pay attention to the safety warnings, which are marked with Warning and Caution.

Safety warnings in these instructions are marked as follows:

	DANGER Draws attention to an immediately threatening danger. Failure to observe this warning will lead to death or very serious injury.
---	---

	WARNING Draws attention to a danger. Failure to observe this warning could lead to death or very serious injury.
---	--

	CAUTION Draws attention to a possibly dangerous situation. Failure to observe this warning could lead to slight injury and/or damage to the product or to damage of property.
---	---

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Icons in this Instruction for use

Icon	Legend
	Note
	Attention Observe special precautions or warnings for this product.

Icons on name plate

Icon	Explanations to type plates
	CE-mark
	Following the instruction for use
	The symbol of the crossed waste container indicates, that Ritter products are electrical equipment according to Directive 2002/96/EC. Within the EU-Countries these products have to be separately disposed. Ritter wants to encourage you, to minimize possible effects of waste to environment and health outside of the EU, too. Please follow the local directives for disposal of waste and make use of the separate collection of electrical equipment.
	Serial number
	Manufacturer/Date of manufacturing
	Medical device

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Fundamentals

The GIRARDELLI X-3D automatic daylight x-ray processor has been developed so that it is ergonomic and versatile in use and easy to operate. The integrated daylight attachment with inclined viewing window makes it possible to develop without a dark-room. The x-ray processor has to be operated in combination with the original Ritter developer and fixer concentrate.

Intended use

The automatic x-ray processor is for the development of intra- and extra-oral x-ray films in combination with the associated Ritter developer and fixer concentrates. It may not be used in the open air or in areas where there might be a danger of explosion. It may only be operated through instructed practice personnel. Pregnant and breastfeeding women and people under 18 years may not operate the appliance.

The GIRARDELLI X-3D automatic daylight x-ray processor is a medical product in accordance with regulation 2017/745 appendix VIII of the European Parliament and of the council concerning medical products and is classified as Class I. The operator has to make sure that the product is operated in accordance with these Instruction for use. The operator is obligated to prepare a hazard analysis and for the used concentrates a user's guide. The operator is responsible to perform a check of the functionality to guarantee a continuous good developing quality.

Functional description

The housing of the GIRARDELLI X-3D automatic daylight x-ray processor is made of blue-grey and white impact-resistant polystyrene. The appliance has an electronic temperature regulator for developer and fixer bath, automatic programme sequence and film drying and makes possible the development of intra-oral films.

The design of the GIRARDELLI X-3D automatic daylight x-ray processor enables even manual development. If a rapid diagnosis is required processing can be interrupted and later continued.

Notes

When imported into countries whose native language is not English or German, the operating manual needs to be translated into local languages.

In this translation, replace the words „original operating manual“ by the following notice: „translation of the original operating manual“.

The Ritter® Concept GmbH is not directly or indirectly liable for mistakes or failures in this documentation as well for the given benefits of the use with this operating instruction!

The illustrations in this manual may differ from the real device.

The duplicating of this documentation without written permission of Ritter® Concept GmbH is prohibited!

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**DANGER - Danger of explosion -**

The appliance may not be used in places where there might be a danger of explosion. The automatic x-ray processor can be used without danger in rooms where inflammable anaesthetics and skinning or skin-disinfecting agents are used according to regulations. It is not permitted to use the appliance when other inflammable materials (e. g. petrol) are present.

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**WARNING - Danger to persons -**

Every time before beginning work connecting cables must be checked for possible damage. Damaged cables must be replaced at once by original manufacturer's cables. Safety notices and certificates are no longer valid if other makers' cables are used.

You must not open the appliance yourself or try to service it yourself as this could invalidate any guarantee claim. The lower part may only be serviced by an authorised technician.

The appliance must not be used, if:

- liquid has penetrated into the lower part, or
- it has fallen and the housing has been damaged.

Since the appliance is not specially protected against liquid penetration it must not be dipped in any liquid.

Pull out the mains plug before cleaning the power pack housing. Always pull on mains plug and not on cable. For this the fitting plug has to be easily accessible. For cleaning use a slightly moistened cloth.

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**WARNING - Functional breakdowns of the appliance -**

Magnetic and electrical fields can adversely affect the function of the appliance. When using it, take care that all machines being operated in the vicinity accord with the relevant EMC requirements. X-ray machines, tomograph machines, radio plants, mobile phones, etc. disturb operation because they are permitted to produce higher levels of electro-magnetic disturbance.

Place the GIRARDELLI X-3D automatic x-ray processor at a sufficient distance from such machines and carry out a functions check before use.

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	<p>WARNING - Danger of suffocation - Store the packing material where it is not accessible to children.</p>
---	--

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	<p>CAUTION - Danger to persons, damage to the appliance - Take care that the instructions regarding surrounding conditions are adhered to. Before cleaning the appliance switch it off and take out the mains plug. For this the fitting plug has to be easily accessible. Do not place anything on the cables: Lay all cables so that they will not cause any risk of stumbling.</p> <p>The baths must not be tipped if there is any liquid in them. Do not spill any liquid near the air dryer. In either case liquid could penetrate into the appliance.</p>
---	--

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	<p>CAUTION - Damage to the appliance - Place the appliance only on a stable, level surface.</p> <p>Do not exert excessive pressure on the appliance since doing so could cause damage.</p> <p>The automatic x-ray processor can be safely stored in a place with a temperature range of -20°C to +60°C (-4°F to +140°F). Permanent damage can be caused if this temperature range is not adhered to. Always store the appliance in the original carton.</p>
---	--

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	<p>DANGER - Damage to health and to the environment - When using developer and fixer concentrates there can be danger for humans and environment. It is mandatory to follow the hazard notes and safety precautions according to appendix of these Operating Instructions (Safety Data Sheet).</p>
---	---

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	<p>DANGER -- Danger to persons, damage to the appliance - During operation, do not interfere with the ongoing development process inside the device. Interfering with the moving parts may result in injuries and damage to the device.</p>
---	--

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	<p>NOTE - serious incidents All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is registered.</p>
---	--

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Transport Damages

General valid: Store goods and packaging unchanged, do not use defective product and do not send back before consultation. Please follow instructions below carefully. Should those not been adhered to, the damage is considered as emerged after the shipment.

Apparent Transport Damages

Apparent transport damages have to be immediately recorded by the forwarder in writing when receiving goods from forwarder/delivery service. In case the transport has been commissioned by Ritter Concept GmbH, send the written confirmation to Ritter Concept GmbH without delay. In case of commissioning of the goods through the recipient, the respective forwarder has to be immediately informed.

Should you have already accepted the goods, immediately notify the damage to the forwarder in writing with description of the issue and apply for inspection. Should the transport have been commissioned by Ritter Concept GmbH, report damage in advance and in writing to Ritter Concept GmbH and send the written confirmation (after inspection of damage) received from the forwarder, immediately to Ritter Concept GmbH.

Hidden Transport Damages

When hidden transport damages appear, inform the forwarder immediately in writing, latest on the 7th day after receipt of goods. Should the transport have been commissioned by Ritter Concept GmbH, also please inform Ritter Concept GmbH in writing immediately.

Unpacking

Prepare a suitable working surface for the appliance before unpacking it. You need a stable, level surface near a mains socket.

- Take the appliance out of the carton and remove the adhesive strips which are a protection against transport damage.
- After unpacking check that all the parts are present and undamaged.

The standard design includes the following separate parts:

• 1 film clip	• 1 top part
• 1 transport shaft, red	• 1 lower part
• 1 transport shaft, black	• 3 bath tanks
• 1 status development basket	• 2 bath lids
• 1 developing concentrate	• 1 mains cable
• 1 fixing concentrate	

Contact the manufacturer or dealer if you discover that any separate parts are missing or damaged.
- Save the packing material and carton for possible return or storage.

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Disposal of Packaging

Valid for Germany: Ritter Concept GmbH is according to regulation on packaging to the taking back system connected. The packaging can be handed over to each public collection point for disposal.

Set-up location

- Ensure that there is adequate ventilation where the appliance is placed.
- Avoid direct sunlight or other strong light sources.
- Ensure that the ambient conditions accord with the instructions in the Technical Information.
- Place the appliance on a stable, level surface.

Installation

	CAUTION As long as there is no liquid in the appliance the bath heating temperature regulator must always be set at 20°C (68°F).
---	--

The bath tanks are in the appliance when it is delivered so that it is only necessary to insert the respective transport shaft (→ see page 14 of this manual), fill the baths and insert the mains plug. Please note that the fitting plug has to be easily accessible at all times.

For safety reasons we recommend you to run the appliance through an residual-current circuit breaker (RCCB).

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Used chemicals

	<p>CAUTION</p> <p>The developer and fixing liquids are only approved for use with Girardelli dip developers.</p>
---	---

Girardelli Entwickler- und Fixierungskonzentrat (EG00000007)
Girardelli developer and fixing liquids (EG00000007)

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 Ritter Concept GmbH draws attention to the following regarding the ingredients of the chemicals used in the developing and fixing liquids (EG00000007):

According to current knowledge

- Substances of the so-called „Candidate List“, according to Article 57, of Regulation (EC) No. 1907/2006 („REACH“) as of 19.01.2021
- CMR substances (category 1A or 1B according to Annex VI Part 3 of Regulation (EC) No. 1272/2008)
- Substances with endocrine disrupting properties

are non-constituent ingredients with more than 0.1% by mass.

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**CAUTION**

As long as there is no liquid in the appliance the bath heating temperature regulator must always be set at 20°C (68°F).

When charging the developer and fixer baths the chemicals must not be allowed to mix since this will render them useless. For safety reasons pay attention that, after filling the first bath, the respective bath lids are immediately placed on. The lids are labelled and must not be interchanged!

For developing only the original Ritter concentrate has to be used. We recommend to charge the concentrate after working hours and leave overnight before using the first time. This allows the concentrate to reach the ideal effectiveness.

Only when using liquid concentrate

- 1) Put 0.5 litres of GIRARDELLI X-3D developer concentrate in the developer bath (1st bath on the right) and fill with water of a moderate temperature (approx. 25°C/77°F) just above the height of the mark. Stir concentrate and water with a stick to make sure that they are well mixed. Put the lid on the developer bath.
- 2) Put 0.5 litres of GIRARDELLI X-3D fixer concentrate in the fixer bath (2nd bath from the right) and fill with water of a moderate temperature (approx. 25°C/77°F) just above the height of the mark. Stir concentrate and water with a stick to make sure that they are well mixed. Put the lid on the fixer bath if it is not going to be used at once.
- 3) Take out the tank for the water bath (3rd bath from the right) and fill with cold water just above the height of the mark. Put bath tank back in the appliance.
- 4) The appliance can now be connected to the mains. You will notice operational readiness of the appliance and the standby-mode (bath-heating in working condition → see chapter „Temperature regulation“ at page 17 of this manual) by the illuminated yellow light, which is located on the temperature-scale on the right side of the appliance.
- 5) The temperature regulator on the right-hand side of the appliance is for setting the required developing temperature. The GIRARDELLI X-3D automatic daylight x-ray processor has been so designed that in conjunction with the GIRARDELLI X-3 developing concentrate the ideal developing temperature is about 25°C (77°F). Set the temperature regulator on the right-hand side of the appliance to 25°C (77°F).

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When heating the developer and fixer bath for the first time after installation the appliance needs approx. 2.5 hours with cold liquid to reach this ideal temperature of 25°C (77°F). However, if developer and fixer concentrate are added to water of moderate temperature, the bath heating process can be shortened significantly.

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Sequence of operational procedures

The ideal developing temperature is about 25°C (77°C).

For detailed explanation please read chapter „Filling the baths“ (→ see page 12 point 5 and 6).

- 1) The development takes place with one of the differently coloured transport shafts. The red transport shaft is being used when Kodak films are utilized, the black transport shaft with Agfa films and films with similar quality. The respective transport shaft can be inserted into the appliance very easily via the feather mechanism.
- 2) Before beginning the day's developing work take off the top of the appliance and remove the bath lids. After that fit the top of the appliance back on its bottom part.

To develop films proceed as follows

- 3) Place the film clip/film basket and the exposed film (in the protective cover) in the stacking compartment. Put the top on the lower part of the appliance.
 - 4) Press the green start button; the transport shaft and the dryer blower are now operative.
 - 5) Put your hands through the handling sleeves.
 - 6) Take the film out of the stacking compartment and remove the protective cover. Attach it to the film clip or put it in the film basket.
 - 7) Finally, place the film clip/film basket between the 1st and 2nd tooth of the cog rail on the transport shaft. As the transport shaft is already operative the film carrier sinks at once into the developer bath and the formation of grey fogging on the film is thus avoided.
- 

Should the transport clamps of the film holders become hard due to a longer time of no use, you can soften it while leaving the clamp (not the entire basket) in hot water (overnight; water can cool down) or in slowly boiling water (approx. 2 hours).
- 8) The hands can be taken out of the sleeves as soon as the film is in the developer bath since the film is now protected against possible light intrusion.
 - 9) At the end of or during the drying process the top can be lifted and the film clip/film basket taken out.
 - 10) At the end of the developing process the appliance must be switched off by pressing the start button again.

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 As long as multiple film carriers remain inside the housing, these carriers can be placed onto the transport shaft immediately after each other. Before a film carrier can be placed inside the housing while a film development is in process, this can not happen before the carrier in process has reached the fixer bath (after approx. 2-3 minutes, depending on which transport shaft is being used). Otherwise the film would be exposed.

	<p>CAUTION</p> <p>Pay attention that not more than 3 film clips and not more than 2 status development baskets that have gone through accumulate at the end of the transport shaft. This could lead to damage to the motor.</p>
--	--

 After finishing of the development process the device must be switched off by pressing the start button.

- 11) Film clip and development basket should be rinsed after each developing process.
- 12) After the last developing procedure of a working day the developer and fixer baths should be covered with the individual bath lids. This has considerable influence on the useful life of the developing chemicals. Do not interchange the bath lids!

Shortening of the developing process

For a rapid diagnosis the film clip/film basket can be removed from the middle of the fixer bath (processing time up to this point approx. 2 minutes with red transport shaft, approx. 3 minutes with black transport shaft). Manually dip the film clip/film basket in the water bath to avoid damage to clothing or floor by dripping fixing solution. After the film has been examined the carrier can be put back on the transport shaft where it was taken out so that the film is fully fixed and dried according to the programme and can be stored.

 When taking out or replacing a film clip/film basket, take care that any other film carriers that may be in the appliance have already reached the fixer bath.

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**Emptying and re-charging with developing
concentrate and water**

WARNING

If the appliance is not used for a lengthy period of time (e.g. holiday) it is essential that it remains disconnected from the mains since over-heating of the baths heater may result.

The mains plug must also be taken out when emptying baths.

Pay attention to always pull on mains plug and not on cable.

CAUTION

If you do not re-fill the baths immediately after emptying it is essential that the temperature regulator for the bath heating is set at 20°C (68°F). Otherwise over-heating can occur.

- 1) To empty the used concentrate take the baths out of the appliance and empty the liquid into suitable disposal canisters.
 - 2) The water bath, which should be regularly emptied after 1-2 weeks, can be emptied into the waste water system.
Before the appliance is filled with fresh concentrate all marks on the appliance and in or on the transport shaft caused by the developing concentrate should be cleaned off. The mains plug must be disconnected when this is done.
Further instructions concerning the cleaning of the device: → see page 18 of this manual
- Procedure for filling and re-charging with developing concentrate and filling the water bath:
→ see page 12 of this manual
- 3) Used concentrate must be disposed of in a proper manner.
→ see page 22 of this manual

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Temperature regulation

The heating process of the baths after charging the concentrate is visible through the permanently illuminated orange control light. Short before the adjusted temperature is reached, the orange light will start blinking in short intervals.

The respective current temperature in the baths is measured by using a thermometer!

It is important that the appliance is always under current entry during day and night. This way the bath heating is always in 'Stand-by' mode, this means that the developer and fixer bath temperature is always constant. Only when the mains plug is disconnected (yellow light on temperature-scale is not illuminated) for lengthy periods of time (e.g. holidays) will the baths cool to room temperature. After re-insertion of the mains plug, the two baths with cold liquid will need a certain while in order to heat up and reach the ideal developing temperature of 25°C (77°F) again.

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 **Do not use cleaning agents containing abrasives or solvents since they could damage the plastic (polystyrene).**

- 1) The appliance must be inoperative and mains plug has to be removed before cleaning the appliance. Always pull on mains plug and not on cable.

When changing the developing concentrate and doing the associated cleaning all marks in the appliance and around the transport shaft caused by the developing concentrate should be cleaned off:

- 2) Bath tanks and appliance should be cleaned with a suitable water-soluble broad-spectrum cleaning concentrate.
- 3) To remove the transport shaft for cleaning, push it to the left and remove it from the carrier (black to the right).

Wet the transport shaft with the cleaning solution and clean the shaft under warm running water by using a soft or middle-hard brush until the grooves in the shaft are completely clean.

This cleaning and maintenance procedure must be carried out regularly every 1-2 weeks. Only this way a smooth transport of the film holders is guaranteed.

- 4) The film clips/film baskets should be rinsed after every working session. They can periodically be cleaned from stains by using the cleaning solution.
- 5) For cleaning of the mains cable, wipe cable with soap water, for disinfection with ethanol-based disinfecting solution. Never dip cable into liquids!
- 6) If necessary, remove the dust with a brush, which has accumulated at the lattice of the air outlet in the drying chamber.

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This guide for recognising and removing faults assumes that the appliance and the films are basically in perfect condition. The reason for the problems/fault should be checked in the order in which they are given below:

Problem/fault	Possible cause/s	Troubleshooting
Increased fogging	developer temperature too high	set temperature to 25°C (77°F)
	light intrusion	avoid light, possibly re-locate the appliance
	faulty film storage	keep films dry, locked and never store under extreme conditions (temperature, etc)
	fixer in developer bath	empty developer bath and re-charge
	fixer depleted	empty fixer bath and re-charge
	developer depleted	empty developer bath and re-charge
Reduced contrast/ Sensitivity	developer temperature too low	set temperature to 25°C (77°F)
	developer wrongly prepared	empty developer bath and re-charge (→ see page 12/16 of this manual)
	developer depleted	empty developer bath and re-charge
	fixer in developer bath	empty developer bath and re-charge
	fixer depleted	empty fixer bath and re-charge
	light intrusion	avoid light, possibly re-locate the appliance
	faulty film storage	keep films dry, locked and never store under extreme conditions (temperature, etc)
Heightened contrast/ Sensitivity	developer temperature too high	set temperature to 25°C (77°F)
	developer wrongly prepared	empty developer bath and re-charge (→ see page 12/16 of this manual)
	fixer in developer bath	empty developer bath and re-charge
	fixer depleted	empty fixer bath and re-charge
	faulty film storage	keep films dry, locked and never store under extreme conditions (temperature, etc)
	light intrusion	avoid light, possibly re-locate the appliance
Wet or damp film	fixer depleted	empty fixer bath and re-charge
	developer depleted	empty developer bath and re-charge
Foggy or milky film	fixer depleted	empty fixer bath and re-charge
Scratches on the films	faulty handling/storage/material	- use original GIRARDELLI X-3D developing concentrate and high-quality x-ray films - store film where dry, not under extreme conditions
	fixer depleted	empty fixer bath and re-charge
	developer depleted	empty developer bath and re-charge
	polluted water	empty water bath and re-charge

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Support

In the case of problems you cannot fix yourself, contact your dealer or the manufacturer (contact information: → see cover sheet)

Repairs

Appliances for repair must be sent to the manufacturer, in the original packaging, free of postal charges for the manufacturer and with quoting production number (serial number), type-model, and giving a description of the fault (contact information: → see cover sheet).

Warranty

In case of no other agreement, a warranty period of 12 months is applied. The period of time starts with the invoice date issued by the manufacturer.

Defects are to be disclosed immediately in written form to the manufacturer after receipt of goods, respectively after detection. The customer is not authorized to ship back products to the manufacturer without prior approval. The customer must choose the cheapest method of transportation.

The manufacturer will not take over the transport costs for a re-delivery, if the destination of the goods differs from the original delivery address.

No warranty claims apply in the following cases:

- Use of the product outside of its conventional application or operation of the product outside its electrical specifications.
- If instructions for use, maintenance and service and connection were not followed.
- Incorrect, insufficient and faulty use through the customer.
- Usage of different developing concentrate, instead of original GIRARDELLI X-3D developing concentrate.
- Modifications through customer.
- Normal wear and tear as well as damages, deficiencies, loss of efficiency or other decline of product characteristics, which lead back to outside influences (e.g. crush, bash, heavy movement, water, fire), wrong storage, use or set-up, abnormal climatic surroundings, special circumstances during receipt, operating conditions at installation site or act of god

For all other cases which are not mentioned, the general terms and conditions of Ritter Concept GmbH apply.

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Disposal

Disposal of the appliance and the used developing concentrate must be carried out by a licensed firm of waste disposers in accordance with the appropriate local regulations and general legal requirements.

For Germany valid: Ritter Concept GmbH is according to regulation on packaging to the taking back system connected. The packaging can be handed over to each public collection point for disposal.

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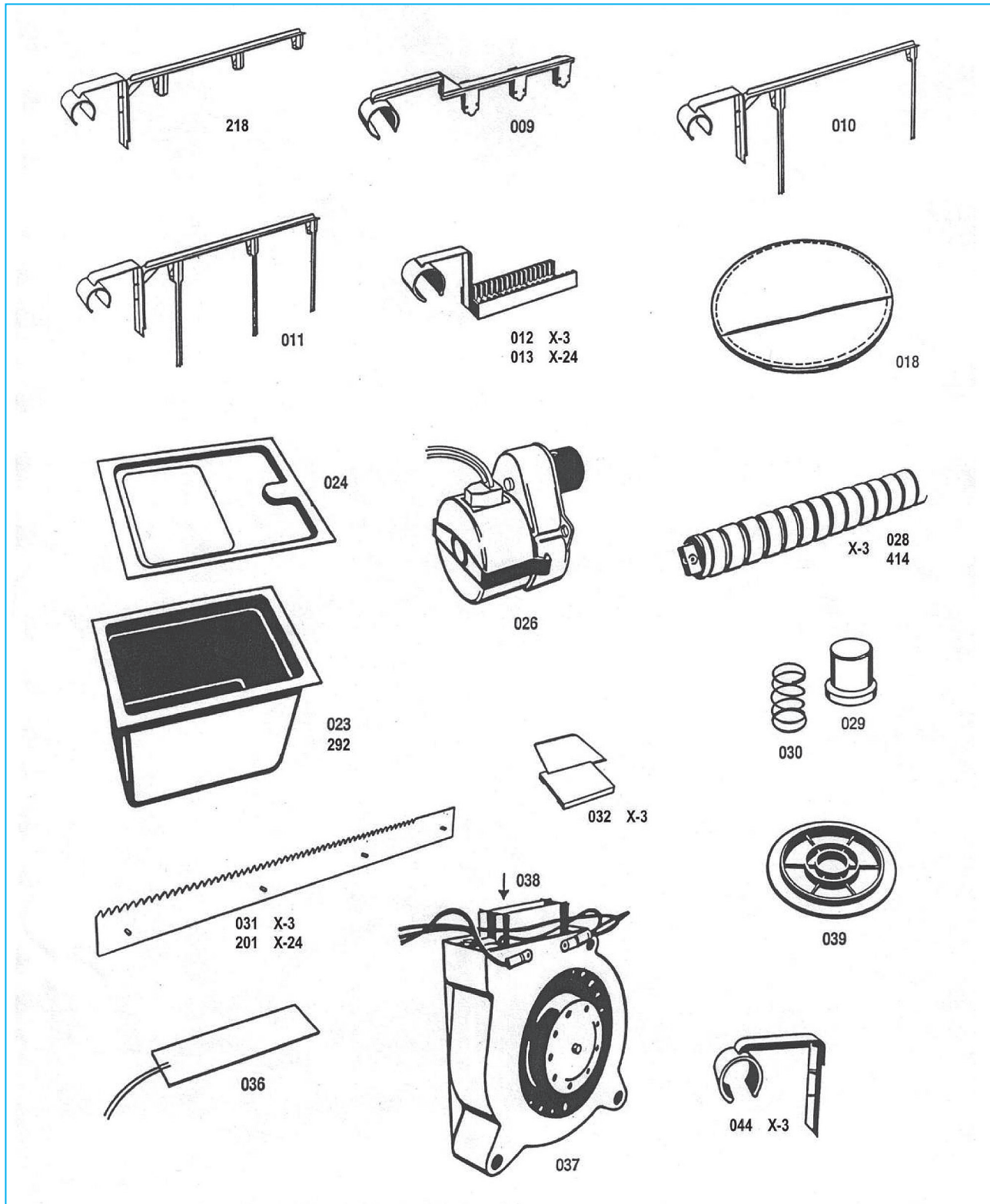
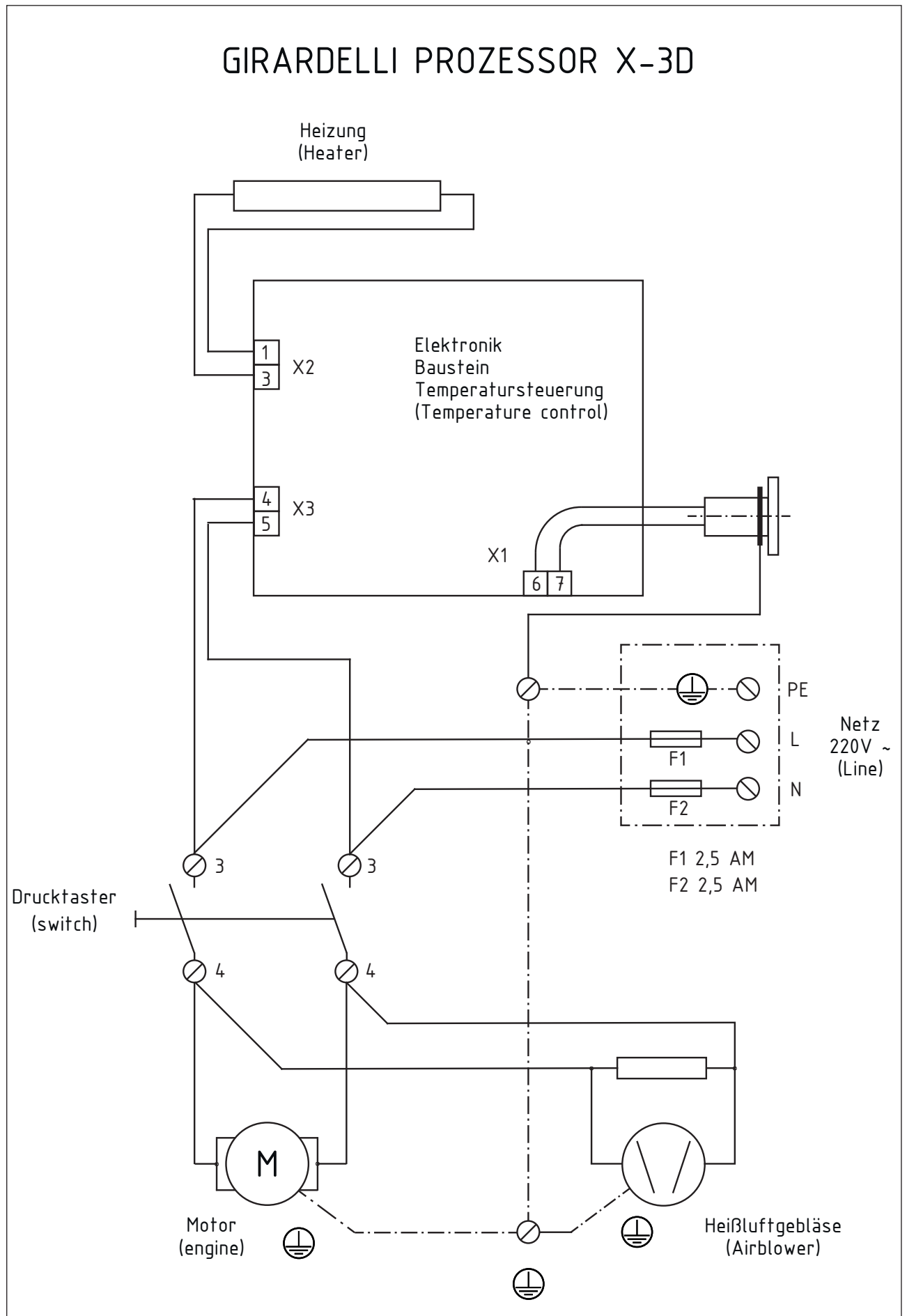
Handelsname: GIRARDELLI X-3D Entwickler

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Trade name: GIRARDELLI X-3D Developer

(Contd. of page 1)

- Hazard pictograms
-    
- GHS05 GHS07 GHS08 GHS09

· Signal word Danger

· Hazard-determining components of labelling:

hydroquinone

H315 Causes skin irritation.

H318 Causes serious eye damage.

H317 May cause an allergic skin reaction.

H341 Suspected of causing genetic defects.

H351 Suspected of causing cancer.

H400 Very toxic to aquatic life.

· Precautionary statements

P273 Avoid release to the environment.

P280 Wear protective gloves / eye protection.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P310 Immediately call a POISON CENTER/doctor.

P333+P313 If skin irritation or rash occurs: Get medical advice/attention.

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

· 2.3 Other hazards

· Results of PBT and vPvB assessment

· PBT: Not applicable.

· vPvB: Not applicable.

SECTION 3: Composition/information on ingredients

- 3.2 Mixtures
- Description: Mixture of the substances listed below with harmless additions.

· Dangerous components:

CAS: 123-31-9	hydroquinone	≥3-55%
EINECS: 204-617-8	⚠ Muta. 2, H341; Carc. 2, H351	
Index number: 604-005-00-4	⚠ Eye Dam. 1, H318	
Reg.nr.: 01-2119524016-51-xxxx	⚠ Aquatic Acute 1, H400 (M=10)	
	⚠ Acute Tox. 4, H302; Skin Sens. 1, H317	
CAS: 139-89-9	Hydroxyethylenediaminetriacetic acid, trisodium salt, solution	0.5-2%
EINECS: 205-381-9	⚠ Skin Irrit. 2, H315; Eye Irrit. 2, H319	
Reg.nr.: 01-2119972845-22-xxxx	⚠ potassium hydroxide	
CAS: 1310-58-3	⚠ Met. Corr. 1, H290; Skin Corr. 1A, H314	≥0.5-2%
EINECS: 215-181-3	⚠ Acute Tox. 4, H302	
Index number: 019-002-00-8		
Reg.nr.: 01-2119487136-33-xxxx	4-(hydroxymethyl)-4-methyl-1-phenylpiperazolidin-3-one	≥0.25-0.5%
CAS: 13047-13-7	⚠ Aquatic Chronic 2, H411	
EINECS: 235-920-3	⚠ Acute Tox. 4, H302; Skin Sens. 1, H317	

· Additional information For the wording of the listed hazard phrases refer to section 16.

(Contd. on page 3)

SECTION 1: Identification of the substance/mixture and of the company/undertaking

· 1.1 Product identifier

· Trade name: GIRARDELLI X-3D Developer

· 1.2 Relevant identified uses of the substance or mixture and uses advised against

To this day we do not have any information about the identified use at the moment. These data are available we will add these to the safety data sheet.

· Application of the substance / the mixture

Photochemicals

Developer for X-ray films

Reserved for industrial and professional use.

· 1.3 Details of the supplier of the safety data sheet

· Manufacturer/Supplier:

Ritter Concept GmbH

Freiburger Str. 45

88400 Biberach

Tel. (07351) 52 925-10

Fax (07351) 52 925-11

e-mail: info@ritterconcept.com

· Informing department:

Tel. (07351) 52 925-10

e-mail: info@ritterconcept.com

· 1.4 Emergency telephone number:

During office hours:

Tel. (07351) 52 925-10

SECTION 2: Hazards identification

· 2.1 Classification of the substance or mixture

· Classification according to Regulation (EC) No 1272/2008

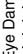
 GHS08 health hazard

Muta. 2 H341 Suspected of causing genetic defects.

Carc. 2 H351 Suspected of causing cancer.

 GHS05 corrosion

Eye Dam. 1 H318 Causes serious eye damage.

 GHS09 environment

Aquatic Acute 1 H400 Very toxic to aquatic life.

 GHS07

Skin Irrit. 2 H315 Causes skin irritation.

Skin Sens. 1 H317 May cause an allergic skin reaction.

· 2.2 Label elements

· Labelling according to Regulation (EC) No 1272/2008

The product is classified and labelled according to the CLP regulation.

(Contd. on page 2)

08

Trade name: GIRARDELLI X-3D Developer

(Contd. of page 3)

See Section 13 for information on disposal.

SECTION 7: Handling and storage

- **7.1 Precautions for safe handling**
Do not handle until all safety precautions have been read and understood.
No special precautions necessary if used correctly.
- **Information about protection against explosions and fires:**
No special measures required.
The product is not flammable
- **7.2 Conditions for safe storage, including any incompatibilities**
- **Storage**
Requirements to be met by storerooms and containers:
Store only in unopened original containers.
Keep container tightly closed in a cool, well-ventilated place.
- **Information about storage in one common storage facility:**
Keep away from foodstuffs, beverages and food.
Do not store together with acids.
Store away from oxidising agents.
- **Further information about storage conditions:**
Protect from heat and direct sunlight.
- **Recommended storage temperature:** 5-25°C
- **Storage class**
LGK 12
(German Technical Rule for Hazardous Substance – TRGS 510)
- **7.3 Specific end use(s)** No further relevant information available.

SECTION 8: Exposure controls/personal protection

- **Additional information about design of technical systems:** No further data; see section 7.
- **8.1 Control parameters**
- **Components with limit values that require monitoring at the workplace:**

123-31-9 hydroquinone	
WEL Long-term value: 0.5 mg/m ³	
1310-58-3 potassium hydroxide	
WEL Short-term value: 2 mg/m ³	
- **Additional information:** The lists that were valid during the compilation were used as basis.
- **8.2 Exposure controls**
- **Personal protective and hygienic measures**
General protective and hygienic measures should be adhered to general rules for handling chemicals.
Keep away from foodstuffs, beverages and food.
Do not eat, drink or smoke while working.
Take off immediately all contaminated clothing.
Wash contaminated clothing before reuse.
Wash hands during breaks and at the end of the work.
Avoid contact with the eyes and skin.
- **Breathing equipment:** Not required.
- **Protection of hands:**
Protective gloves.
The protective gloves to be used must comply with the specifications of the EC directive 89/686/EEC and the resultant standard EN 374.

(Contd. on page 5)

Trade name: GIRARDELLI X-3D Developer

(Contd. of page 2)

SECTION 4: First aid measures

- **4.1 Description of first aid measures**
- **General information**
Personal protection for the First Aider.
Instantly remove any clothing soiled by the product.
- **After inhalation** Supply fresh air.
- **After skin contact** Instantly wash with water and soap and rinse thoroughly.
- **After eye contact**
Rinse opened eye for several minutes under running water (at least 15 minutes).
Remove contact lenses, if present and easy to do.
Use eye protection.
Call a doctor immediately.
- **After swallowing**
Rinse out mouth and then drink plenty of water.
Do not induce vomiting; instantly call for medical help.
- **4.2 Most important symptoms and effects, both acute and delayed**
No known symptoms to date.
- **4.3 Indication of any immediate medical attention and special treatment needed**
No further relevant information available.

SECTION 5: Firefighting measures

- **5.1 Extinguishing media**
- **Suitable extinguishing agents**
CO2, extinguishing powder or water jet. Fight larger fires with water spray jet or alcohol-resistant foam.
Use fire fighting measures that suit the environment.
- **For safety reasons unsuitable extinguishing agents** Water with a full water jet.
- **5.2 Special hazards arising from the substance or mixture**
Can be released in case of fire
- **5.3 Advice for firefighters**
Sulphur oxides (SOX)
Carbon monoxide
- **Protective equipment:**
Do not inhale explosion gases or combustion gases.
At formation of toxic gases:
Put on breathing apparatus.
- **Additional information** The product is not flammable

SECTION 6: Accidental release measures

- **6.1 Personal precautions, protective equipment and emergency procedures**
Wear protective clothing.
- **6.2 Environmental precautions:** Do not allow to enter drainage system, surface or ground water.
- **6.3 Methods and material for containment and cleaning up:**
Absorb with liquid-binding material (sand, diatomite, acid binders, universal binders, sawdust).
Dispose of contaminated material as waste according to item 13.
Small amounts may be diluted with plenty of water and washed away. Dispose of bigger amounts in accordance with Local Authority requirements.
- **6.4 Reference to other sections**
See Section 8 for information on personal protection equipment.

Safety data sheet
according to 1907/2006/EC, Article 31

Printing date 09.04.2019 Version 7 Revision: 09.04.2019

Trade name: GIRARDELLI X-3D Developer

9.2 Other information No further relevant information available. (Contd. of page 5)

SECTION 10: Stability and reactivity

- **10.1 Reactivity** No further relevant information available.
- **10.2 Chemical stability**
- **Thermal decomposition / conditions to be avoided:**
No decomposition if used according to specifications.
- **10.3 Possibility of hazardous reactions**
Reacts with strong acids
Reacts with acids releasing sulphur dioxide
- **10.4 Conditions to avoid** No further relevant information available.
- **10.5 Incompatible materials:** No further relevant information available.
- **10.6 Hazardous decomposition products:** No dangerous decomposition products known

SECTION 11: Toxicological information

- **11.1 Information on toxicological effects**
- **Acute toxicity** Based on available data, the classification criteria are not met.
- **LD/LC50 values that are relevant for classification:**
- Oral | LD50 | 320 mg/kg
- Oral | LD50 | 365 mg/kg (rat)

- **Primary irritant effect:**
- **Skin corrosion/irritation**
Causes skin irritation.
- **Serious eye damage/irritation**
Causes serious eye damage.
- **Respiratory or skin sensitisation**
May cause an allergic skin reaction.
- **Additional toxicological information:**
At present there are no animal experimental data.
This statement was deduced from the properties of the single components.
- **CMR effects (carcinogenicity, mutagenicity and toxicity for reproduction)**
- **Germ cell mutagenicity**
Suspected of causing genetic defects.
- **Carcinogenicity**
Suspected of causing cancer.
- **Reproductive toxicity** Based on available data, the classification criteria are not met.
- **STOT-single exposure** Based on available data, the classification criteria are not met.
- **STOT-repeated exposure** Based on available data, the classification criteria are not met.
- **Aspiration hazard** Based on available data, the classification criteria are not met.

SECTION 12: Ecological information

- **12.1 Toxicity**
- **Aquatic toxicity:**
Not determined

(Contd. on page 7)
GB

Safety data sheet
according to 1907/2006/EC, Article 31

Printing date 09.04.2019 Version 7 Revision: 09.04.2019

Trade name: GIRARDELLI X-3D Developer

This recommendation applies only to the product stated in the Safety Data Sheet and supplied by us as well as to the purpose specified by us.
Only use chemical-protective gloves with CE-labelling of category III.

Selection of the glove material on consideration of the penetration times, rates of diffusion and the degradation

- **Material of gloves** Breakthrough time (mm) (min)
- Nitril rubber 0.38 > 480
- Neoprene 0.65 > 240
- Butyl rubber 0.36 > 480
- Avoid natural rubber gloves.
- **As protection from splashes gloves made of the following materials are suitable:**
Synthetic gloves
- Value for permeation: Level:
≥ 3 (60 min)
- **Eye protection:** Safety glasses
- **Body protection:** Protective work clothing.

(Contd. of page 4)

SECTION 9: Physical and chemical properties

- **9.1 Information on basic physical and chemical properties**
- **General Information**
- **Appearance:**
- **Form:** Fluid
- **Colour:** Light yellow
- **Odour:** Not characteristic
- **pH-value at 25 °C:** 11.1
- **Change in condition** < 5 °C
- **Melting point/freezing point:** < 5 °C
- **Initial boiling point and boiling range:** > 100 °C
- **Flash point:** Not applicable
- **Self-inflammability:** Product is not selfigniting.
- **Explosive properties:** Product is not explosive.
- **Critical values for explosion:**
- **Oxidising properties** None
- **Vapour pressure at 20 °C:** 23 hPa
- **Density at 20 °C** 1.29 g/cm³
- **Solubility in / Miscibility with Water:** miscible
- **Viscosity:** Not determined
- **dynamic:** Not determined
- **kinematic:** Not determined
- **Solvent content:**
- **Organic solvents:** 0.0 %
- **Water:** ~ 65 %

(Contd. on page 6)
GB

Trade name: GIRARDELLI X-3D Developer (Contd. of page 7)

<ul style="list-style-type: none"> · 14.3 Transport hazard class(es) · ADR/RID, IMDG, IATA  <ul style="list-style-type: none"> · Class 9 · Label 9 <p>9 Miscellaneous dangerous substances and articles.</p>
<ul style="list-style-type: none"> · 14.4 Packing group ADR/RID, IMDG, IATA <p>III</p>
<ul style="list-style-type: none"> · 14.5 Environmental hazards: · Marine pollutant: · Special marking (ADR/RID): · Special marking (ATA): · 14.6 Special precautions for user <p>Product contains environmentally hazardous substances: hydroquinone Symbol (fish and tree) Symbol (fish and tree) Symbol (fish and tree) Warning: Miscellaneous dangerous substances and articles.</p>
<ul style="list-style-type: none"> · 14.7 Transport in bulk according to Annex II of Marpol and the IBC Code <p>Not applicable.</p>
<ul style="list-style-type: none"> · Transport/Additional information: <p>These substances when transported in single or combination packagings containing a net quantity per single or inner packaging of 5 l or less for liquids or having a net mass of 5 kg or less for solids, are not subject to any other provisions of these Regulations provided the packagings meet the general See the following notes. Goods are not subject to the provisions in accordance with the special provision 375 ADR. Goods are not subject to the provisions in accordance with 2.10.2.7 IMDG-Code. Goods are not subject to the provisions in accordance with the special provision A197 IATA-DGR.</p>
<ul style="list-style-type: none"> · ADR/RID · IMDG · IATA · UN "Model Regulation": <p>UN 3082 ENVIRONMENTALLY HAZARDOUS SUBSTANCE, LIQUID, N.O.S. (HYDROQUINONE), 9, III</p>

SECTION 15: Regulatory information

- 15.1 Safety, health and environmental regulations/legislation specific for the substance or mixture
- Directive 2012/18/EU
- Named dangerous substances - ANNEX I None of the ingredients is listed.
- Seveso category E1 Hazardous to the Aquatic Environment
- Qualifying quantity (tonnes) for the application of lower-tier requirements 100 t
- Qualifying quantity (tonnes) for the application of upper-tier requirements 200 t
- REGULATION (EC) No 1907/2006 ANNEX XVII Conditions of restriction: 3

(Contd. on page 9) GB

Trade name: GIRARDELLI X-3D Developer (Contd. of page 6)

<ul style="list-style-type: none"> · 123-31-9 hydroquinone <p>EC50/48 h 0.29 mg/l (Daphnia magna) IC50/72 h 0.335 mg/l (Selenastrum capricornutum) LC50/96 h 0.044 mg/l (Pimephales promelas)</p>
<ul style="list-style-type: none"> · 584-08-7 potassium carbonate <p>EC50/48 h 380-820 mg/l (Daphnia magna) LC50/96 h 310-750 mg/l (Pimephales promelas)</p>
<ul style="list-style-type: none"> · 12.2 Persistence and degradability · Behaviour in environmental systems · 12.4 Mobility in soil · Ecotoxicological effects · Behaviour in sewage processing plants · Additional ecological information: · General notes: · 12.5 Results of PBT and vPvB assessment · vPvB: · 12.6 Other adverse effects <p>Not determined Not determined No further relevant information available. No further relevant information available. Not determined Not determined At present there are no ecotoxicological assessments. This statement was deduced from the properties of the single components. Water hazard class 3 (German Regulation) (Self-assessment): highly water-endangering. Do not allow product to reach ground water, water bodies or sewage system. Not applicable. Not applicable. No further relevant information available.</p>

SECTION 13: Disposal considerations

- 13.1 Waste treatment methods
- Recommendation
- European waste catalogue
- Uncleaned packagings:
- Recommendation:
- Recommended cleaning agent:

Must not be disposed of together with household garbage. Do not allow product to reach sewage system.
 Must be specially treated under adherence to official regulations.
 09 01 01* water-based developer and activator solutions
 Non contaminated packagings can be used for recycling. Empty contaminated packagings thoroughly. They can be recycled after thorough and proper cleaning.
 Water, if necessary with cleaning agent.

SECTION 14: Transport information

- 14.1 UN-Number
- ADR/RID, IMDG, IATA
- 14.2 UN proper shipping name
- ADR/RID, IMDG, IATA

UN 3082
 ENVIRONMENTALLY HAZARDOUS SUBSTANCE, LIQUID, N.O.S. (hydroquinone)

(Contd. on page 8) GB

Page 9/10

Safety data sheet
according to 1907/2006/EC, Article 31

Printing date 09.04.2019

Version 7

Revision: 09.04.2019

Trade name: GIRARDELLI X-3D Developer

(Contd. of page 8)

- **National regulations**

- **Information about limitation of use:**

Employment restrictions concerning pregnant and lactating women must be observed.

Class	Share in %
I	3,9

- **Water hazard class:** Water danger class 3 (Self-assessment): highly water-endangering.
- **15.2 Chemical safety assessment:** A Chemical Safety Assessment has not been carried out.

SECTION 16: Other information

These data are based on our present knowledge. However, they shall not constitute a guarantee for any specific product features and shall not establish a legally valid contractual relationship.

- **Relevant phrases**

- H290 May be corrosive to metals.
- H302 Harmful if swallowed.
- H314 Causes severe skin burns and eye damage.
- H315 Causes skin irritation.
- H317 May cause an allergic skin reaction.
- H318 Causes serious eye damage.
- H319 Causes serious eye irritation.
- H341 Suspected of causing genetic defects.
- H351 Suspected of causing cancer.
- H400 Very toxic to aquatic life.
- H411 Toxic to aquatic life with long lasting effects.

- **Recommended restriction of use** No public product, only for commercial use

- **Classification according to Regulation (EC) No 1272/2008**
The classification of the mixture is generally based on the calculation method using substance data according to Regulation (EC) No 1272/2008.

- **Abbreviations and acronyms:**

RID: Règlement international concernant le transport des marchandises dangereuses par chemin de fer (Regulations Concerning the International Transport of Dangerous Goods by Rail)
ICAO: International Civil Aviation Organisation
ADR: Accord européen sur le transport des marchandises dangereuses par Route (European Agreement concerning the International Carriage of Dangerous Goods by Road)
IMDG: International Maritime Code for Dangerous Goods
IATA: International Air Transport Association
GHS: Globally Harmonised System of Classification and Labelling of Chemicals
EINECS: European Inventory of Existing Commercial Substances
ELCS: European List of Notified Chemical Substances
USDA: Chemical Abstracts Service (division of the American Chemical Society)
LD50: Lethal dose: 50 percent
PBT: Persistent, Bioaccumulative and Toxic
vPvB: very Persistent and very Bioaccumulative
Met. Corr.1: Corrosive to metals – Category 1
Acute Tox. 4: Acute toxicity – Category 4
Skin Corr.-1A: Skin corrosion/irritation – Category 1A
Skin Irrit. 2: Skin corrosion/irritation – Category 2
Eye Dam. 1: Serious eye damage/eye irritation – Category 1
Eye Irrit. 2: Serious eye damage/eye irritation – Category 2
Skin Sens. 1: Skin sensitisation – Category 2
Min. G. 2: Carcinogenicity – Category 2
Aquat. Acute 1: Hazardous to the aquatic environment - acute aquatic hazard – Category 1
Aquat. Chronic 2: Hazardous to the aquatic environment - long-term aquatic hazard – Category 2

- **Sources**
applicable EEC directives:

(Contd. on page 10)

GB

Page 10/10

Safety data sheet
according to 1907/2006/EC, Article 31

Printing date 09.04.2019

Version 7

Revision: 09.04.2019

Trade name: GIRARDELLI X-3D Developer

(Contd. of page 9)

- 1907/2006
 - 1272/2008
- Internal physical tests, MSDS of the ingredients, Information system on hazardous substances of the German Social Accident Insurance (GESTIS-database on hazardous substances), <http://www.dguv.de/ifa/en/gestis/stoffdb/index.jsp>
- *** Data compared to the previous version altered.**

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Druckdatum: 09.04.2019

Version 7

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Handelsname: GIRARDELLI X-3D Fixierer

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(Contd. of page 1)

SECTION 4: First aid measures

- **4.1 Description of first aid measures**
- **General information** Instantly remove any clothing soiled by the product.
- **After inhalation** Supply fresh air.
- **After skin contact**
Instantly wash with water and soap and rinse thoroughly.
The product is not skin irritating.
- **After eye contact**
Rinse opened eye for several minutes under running water (> 15 min). Then consult doctor.
- **After swallowing**
Rinse out mouth and then drink plenty of water.
In case of persistent symptoms consult doctor.
- **4.2 Most important symptoms and effects, both acute and delayed**
No known symptoms to date.
- **4.3 Indication of any immediate medical attention and special treatment needed**
No further relevant information available.

SECTION 5: Firefighting measures

- **5.1 Extinguishing media**
- **Suitable extinguishing agents**
CO₂, extinguishing powder or water jet. Fight larger fires with water spray jet or alcohol-resistant foam.
Use fire fighting measures that suit the environment.
- **For safety reasons unsuitable extinguishing agents** Water with a full water jet.
- **5.2 Special hazards arising from the substance or mixture**
Can be released in case of fire
sulphur dioxide (SO₂)
Nitrogen oxides (NO_x)
- **5.3 Advice for firefighters**
- **Protective equipment:**
Do not inhale explosion gases or combustion gases.
Put on breathing apparatus.
- **Additional information** The product is not flammable

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- **6.1 Personal precautions, protective equipment and emergency procedures** Not required.
- **6.2 Environmental precautions:** Do not allow to enter drainage system, surface or ground water.
- **6.3 Methods and material for containment and cleaning up:**
Absorb with liquid-binding material (sand, diatomite, acid binders, universal binders, sawdust).
Dispose of contaminated material as waste according to item 13.
- **6.4 Reference to other sections**
See Section 8 for information on personal protection equipment.
See Section 13 for information on disposal.

SECTION 7: Handling and storage

- **7.1 Precautions for safe handling** No special measures required.
- **Information about protection against explosions and fires:**
No special measures required.

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- **1.1 Product identifier**
- **Trade name:** GIRARDELLI X-3D Fixer
- **1.2 Relevant identified uses of the substance or mixture and uses advised against**
To this day we do not have any information about the identified use at the moment. These data are available we will add these to the safety data sheet.
- **Application of the substance / the mixture**
Photochemicals
Fixing agent for X-ray films
Reserved for industrial and professional use.
- **1.3 Details of the supplier of the safety data sheet**
- **Manufacturer/Supplier:**
Ritter Concept GmbH
Freiburger Str. 45
88400 Biberach
Tel. (07351) 52 925-10
Fax (07351) 52 925-11
e-mail: info@ritterconcept.com
- **Informing department:**
Tel. (07351) 52 925-10
e-mail: info@ritterconcept.com
- **1.4 Emergency telephone number:**
During office hours:
Tel. (07351) 52 925-10

SECTION 2: Hazards identification

- **2.1 Classification of the substance or mixture**
- **Classification according to Regulation (EC) No 1272/2008**
The product is not classified, according to the CLP regulation.
- **2.2 Label elements**
- **Labelling according to Regulation (EC) No 1272/2008** Void
- **Hazard pictograms** Void
- **Signal word** Void
- **Hazard statements** Void
- **Additional information:**
EUH210. Safety data sheet available on request.
- **2.3 Other hazards**
- **Results of PBT and vPvB assessment**
- **PBT:** Not applicable.
- **vPvB:** Not applicable.

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- **3.2 Mixtures**
- **Description:** Mixture of the substances listed below with harmless additions.

CAS: 64-19-7 EINECS: 200-580-7 Index number: 607-002-00-6 Reg.nr.: 01-2119475328-30-xxxx		2-5% Flam. Liq. 3, H226 Skin Corr. 1A, H314 Acute Tox. 4, H312
acetic acid		

- **Additional information** For the wording of the listed hazard phrases refer to section 16.

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· 9.1 Information on basic physical and chemical properties

· General Information

· Appearance:

Form: Fluid

Colourless

· Odour: Not characteristic

· pH-value at 25 °C: 5.3

· Change in condition

Melting point/freezing point: < 5 °C

Initial boiling point and boiling range: > 100 °C

· Flash point: Not applicable

· Self-inflammability: Product is not selfigniting.

· Explosive properties: Product is not explosive.

· Vapour pressure at 20 °C: 23 hPa

· Density at 20 °C: 1.36 g/cm³

· Solubility in / Miscibility with Water: miscible

· Viscosity: Not determined

dynamic: Not determined

kinematic: Not determined

· Solvent content:

Organic solvents: 0 %

Water: ~ 50 %

· 9.2 Other information: No further relevant information available.

SECTION 10: Stability and reactivity

· 10.1 Reactivity: No further relevant information available.

· 10.2 Chemical stability

· Thermal decomposition / conditions to be avoided: No decomposition if used according to specifications.

· 10.3 Possibility of hazardous reactions

Reacts with strong acids and alkali

Reacts with acids releasing sulphur dioxide

Reacts with alkalis releasing ammonia

· 10.4 Conditions to avoid: No further relevant information available.

· 10.5 Incompatible materials: No further relevant information available.

· 10.6 Hazardous decomposition products: Sulphur dioxide

Ammonia

SECTION 11: Toxicological information

· 11.1 Information on toxicological effects

· Acute toxicity: Based on available data, the classification criteria are not met.

· Primary irritant effect:

· Skin corrosion/irritation: Based on available data, the classification criteria are not met.

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(Contd. of page 2)

The product is not flammable

· 7.2 Conditions for safe storage, including any incompatibilities

· Storage

· Requirements to be met by storerooms and containers: No special requirements.

· Information about storage in one common storage facility:

Keep away from foodstuffs, beverages and food.

· Further information about storage conditions: Store in a cool place.

· Recommended storage temperature: 5-25 °C

· Storage class

LGK 12

(German Technical Rule for Hazardous Substance – TRGS 510)

· 7.3 Specific end use(s): No further relevant information available.

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· Additional information about design of technical systems: No further data; see section 7.

· 8.1 Control parameters

· Components with limit values that require monitoring at the workplace:

64-19-7 acetic acid

WEL Short-term value: 50 mg/m³, 20 ppm

Long-term value: 25 mg/m³, 10 ppm

· Additional information: The lists that were valid during the compilation were used as basis.

· 8.2 Exposure controls

· Personal protective equipment

· General protective and hygienic measures

The usual precautionary measures should be adhered to general rules for handling chemicals.

· Breathing equipment: Not required.

· Protection of hands:

Protective gloves.

The glove material has to be impermeable and resistant to the product/ the substance/ the preparation.

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Synthetic gloves

Avoid natural rubber gloves.

· Penetration time of glove material

Value for permeation: Level ≥ 6 (480 min)

The determined penetration times according to EN 374 part III are not performed under practical conditions. Therefore a maximum wearing time, which corresponds to 50% of the penetration time, is recommended.

· As protection from splashes gloves made of the following materials are suitable:

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Revision: 09.04.2019

Trade name: GIRARDELLI X-3D Fixer

Recommended cleaning agent: Water, if necessary with cleaning agent. (Contd. of page 5)

SECTION 14: Transport information

- 14.1 UN-Number: Void
- ADR/RID, ADN, IMDG, IATA: Void
- 14.2 UN proper shipping name: Void
- ADR/RID: Void
- ADN, IMDG, IATA: Void
- 14.3 Transport hazard class(es): Void
- ADR/RID, ADN, IMDG, IATA: Void
- Class: Void
- 14.4 Packing group: Void
- ADR/RID, IMDG, IATA: Void
- 14.5 Environmental hazards: Not applicable.
- 14.6 Special precautions for user: Not applicable.
- 14.7 Transport in bulk according to Annex II of Marpol and the IBC Code: Not applicable.
- Transport/Additional information: Not dangerous according to the above specifications.
- UN "Model Regulation": Void

SECTION 15: Regulatory information

- 15.1 Safety, health and environmental regulations/legislation specific for the substance or mixture
 - Directive 2012/18/EU
 - Named dangerous substances - ANNEX I None of the ingredients is listed.
 - REGULATION (EC) No 1907/2006 ANNEX XVII Conditions of restriction: 65
 - National regulations
- | Class | Share in % |
|-------|------------|
| I | 0.9 |
| II | 4.9 |
- Water hazard class: Water hazard class 1 (Self-assessment); weakly water-endangering.
 - 15.2 Chemical safety assessment: A Chemical Safety Assessment has not been carried out.

SECTION 16: Other information

- These data are based on our present knowledge. However, they shall not constitute a guarantee for any specific product features and shall not establish a legally valid contractual relationship.
- **Relevant phrases**
H226 Flammable liquid and vapour.
H312 Harmful in contact with skin.
H314 Causes severe skin burns and eye damage.
 - **Recommended restriction of use** No public product, only for commercial use

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Revision: 09.04.2019

Trade name: GIRARDELLI X-3D Fixer

- **Serious eye damage/irritation** Based on available data, the classification criteria are not met.
- **Respiratory or skin sensitisation** Based on available data, the classification criteria are not met.
- **Subacute to chronic toxicity**: No known symptoms to date.
- **Additional toxicological information**:
This statement was deduced from the properties of the single components.
- **CMR effects (carcinogenicity, mutagenicity and toxicity for reproduction)**:
Germ cell mutagenicity Based on available data, the classification criteria are not met.
- **Carcinogenicity** Based on available data, the classification criteria are not met.
- **Reproductive toxicity** Based on available data, the classification criteria are not met.
- **STOT-single exposure** Based on available data, the classification criteria are not met.
- **STOT-repeated exposure** Based on available data, the classification criteria are not met.
- **Aspiration hazard** Based on available data, the classification criteria are not met.

SECTION 12: Ecological information

- 12.1 Toxicity
- **Aquatic toxicity**: Not determined
- 12.2 Persistence and degradability: Not determined
- 12.3 Bioaccumulative potential: Not determined
- **Behaviour in environmental systems**: Not determined
- 12.4 Mobility in soil: No further relevant information available.
- **Ecotoxicological effects**: Not determined
- **Behaviour in sewage processing plants**: Not determined
- **Additional ecological information**:
· **General notes**:
Water hazard class 1 (German Regulation) (Self-assessment); Weakly water-endangering.
The product does not contain organically bounded halogens (AOX-free).
At present there are no ecotoxicological assessments.
This statement was deduced from the properties of the single components.
- 12.5 Results of PBT and vPvB assessment
- PBT: Not applicable.
- vPvB: Not applicable.
- 12.6 Other adverse effects: No further relevant information available.

SECTION 13: Disposal considerations

- 13.1 Waste treatment methods
- **Recommendation**
Must not be disposed of together with household garbage. Do not allow product to reach sewage system.
Must be specially treated under adherence to official regulations.
- **European waste catalogue**
09 01 04* fixer solutions
- **Uncleaned packagings**:
· **Recommendation**:
Non contaminated packagings can be used for recycling.
Empty contaminated packagings thoroughly. They can be recycled after thorough and proper cleaning.
Packagings that cannot be cleaned are to be disposed of in the same manner as the product.
EAK-No. 15 01 10

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Trade name: GIRARDELLI X-3D Fixer

(Contd. of page 6)

• **Classification according to Regulation (EC) No 1272/2008**

The classification of the mixture is generally based on the calculation method using substance data according to Regulation (EC) No 1272/2008.

• **Abbreviations and acronyms:**

RID: Règlement international concernant le transport des marchandises dangereuses par chemin de fer (Regulations concerning the International Transport of Dangerous Goods by Rail)
 ICAO: International Civil Aviation Organisation
 ADR: Approbation des marchandises dangereuses par Route (European Agreement concerning the International Carriage of Dangerous Goods by Road)
 IMDG: International Maritime Code for Dangerous Goods
 IATA: International Air Transport Association
 GHS: Globally Harmonised System of Classification and Labelling of Chemicals
 EINECS: European Inventory of Existing Commercial Substances
 ELINCS: European List of Notified Chemical Substances
 CAS: Chemical Abstracts Service (division of the American Chemical Society)
 PBT: Persistent, Bioaccumulative and Toxic
 vPvB: very Persistent and very Bioaccumulative
 Flam. Liq. 3: Flammable liquids – Category 3
 Acute Tox. 4: Acute toxicity – Category 4
 Skin Corr. 1A: Skin corrosion/irritation – Category 1A

• **Sources**

applicable EEC directives:

- 1907/2006

- 1272/2008

Internal physical tests, MSDS of the ingredients,

Information system on hazardous substances of the German Social Accident Insurance (GESTIS-database on hazardous substances) ; <http://www.dguv.de/ifa/en/gestis/stofdb/index.jsp>

• * Data compared to the previous version altered.

68