Description

VivaStyle® is a ready-for-use whitening gel for lightening discoloured teeth. After the first application by the dentist, the product is self-administered by the consumer. The gel is applied by means of a custom-fitted tray.

Composition

Glycerine, urea (carbamide) peroxide, carbomer, tromethamine, potassium nitrate, disodium pyrophosphate, PEG-40 hydrogenated castor oil, sodium stannate, dipotassium EDTA, aroma.

Active ingredient: 16% carbamide peroxide, which corresponds to 5.8% hydrogen peroxide equivalents.

One syringe contains 3.7 g gel. This corresponds to 3 ml.

Areas of use:

- Whitening of discoloured vital teeth
- Whitening of discoloured non-vital teeth

Limitations of use:

VivaStyle Gel must not be used:

- during pregnancy or lactation;
- on a person under 18 years of age;
- by heavy smokers;
- by people who consume large quantities of alcohol;
- by people who are known to be allergic to any of the material's ingredients;
- in the presence of exposed tooth necks;
- in the presence of untreated primary or secondary caries;
- in the presence of restorations with marginal gaps.

Applicable to EU countries:

Contains carbamide peroxide. Avoid contact with eyes.

Rinse immediately if product comes into contact with them. Not to be used on a person under 18 years of age. To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.

Keep out of the reach of children.

Side effects

Temporary sensitivity of the oral mucous membrane or the teeth may occur. These side effects usually subside within a short time after the treatment is finished or discontinued.

VivaStyle® 16%

VivaStyle® 16%

[en] Instructions for Use Tooth whitening gel for at-home applications	[fi] Käyttöohjeet Hammasiden valkaisu geeli kotikäyttöön	[uk] Інструкції для використання Відбілювальний гель зубів для домашнього застосування
[de] Gebrauchsinformation Gel zur Zahnaufhellung für den häuslichen Gebrauch	[no] Bruksanvisning Tannhvitningsgel til hjemmebrauk	[et] Kasutusjuhised Hammasvalgendusgeel koduse kasutamiseks
[fr] Mode d'emploi Gel d'éclaircissement dentaire pour application ambulatoire	[nl] Gebruiksaanwijzing Gel voor het bleken van gebits-elementen voor thuisgebruik	[lv] Lietošanas instrukcija Zobu balināšanas gēls lietošanai mājās apstākļos
[it] Istruzioni d'uso Gel sbiancante per denti per uso domiciliare	[el] Οδηγίες Χρήσεως Γέλ λεύκανσης δοντιών για εφ'ομοίην στο σπίτι	[lt] Naudojimo instrukcija Namie naudoti skirtingai dantų balinimo gelis
[es] Instrucciones de uso Gel de blanqueamiento dental para la aplicación en el domicilio	[tr] Kullanma Talimatı Etilde uygulanan diş beyazlatıcı jel	
[pt] Instruções de Uso Gel para clareamento dental para aplicação caseira	[ru] Инструкции по применению Гель для отбеливания зубов в домашних условиях	
	[mk] Упатства за употреба Гел за белење на забице за апликација дома	
	[bg] Инструкции за употреба Гел за избелване на зъби за домашно приложение	
	[sq] Udhëzime për përdorim Xhel zbardhues dhëmbësh për aplikime në shtëpi	

Instructions for dentists: Before VivaStyle 16% is used, the dentist should carefully examine the patient's medical history, inform the patient about the application procedure and the patient's teeth should be cleaned professionally. It is advisable for the dentist to monitor the whitening process.

- VivaStyle should be used before new tooth-coloured restorations are placed, since restorative materials cannot be whitened. The treatment with VivaStyle should be completed two weeks prior to the placement of tooth-coloured restorations. Existing restorations may not match the appearance of the tooth structure after the whitening process and therefore may have to be replaced.

Recommendation:

A properly fabricated tray, e.g. made of 558805AN VivaStyle Tray Sheets 12 from Ivoclar (not included in the scope of delivery), will enhance the effectiveness of the whitening process.

Well fitted trays will improve patient comfort and reduce the chance of irritation of the mucous membrane occurring.

Instructing the patient:

Careful instruction of the patient is fundamental to the safe and effective application of VivaStyle tooth whitening gel at home.

1. Review each individual step of the application procedure in detail with your patient, using the enclosed information on VivaStyle.
2. Make sure that the tray fits accurately. If necessary, adjust by grinding to prevent any damage to the gums.
3. Using a VivaStyle gel syringe, show the patient how to fill the tray correctly.

Important: Do not overfill the tray and only apply gel on the surfaces where it is needed. The calibration markings on the syringe should be used as a guide when extruding the material.

4. Show the patient how to insert the tray into the mouth. Explain the proper procedure for removing any excess tooth whitening gel.
5. Determine how long the patient should wear the tray. One hour a day is usually sufficient. Depending on the individual requirements of the patient, a longer wear time may be prescribed. The overall application time should not exceed six weeks.
6. Explain the proper removal and cleaning procedures for the tray. The tray should be stored in the accompanying case.

At-home application:

1. Before you apply VivaStyle gel, thoroughly brush and floss your teeth.
2. Remove the cap from the tip of the syringe. Place a small amount of gel toward the facial side of each tooth in the tray. Use the calibration markings on the syringe as a guide when dispensing the material. Do not overfill the gel in the tray. Place the cap back on the application tip.
3. Place the gel-filled tray over your teeth.
Do not squeeze the tray when you insert it.
4. Use a toothbrush or cotton bud to remove excess gel. Do not swallow.
5. Wear the tray as prescribed by your dentist, e.g. for one hour a day.
6. Remove the tray after the prescribed time.
7. Rinse the mouth thoroughly to remove residual gel.
Spit everything out – do not swallow.
8. Clean the tray with a toothbrush under running water. Always store the tray in the accompanying tray case.
9. Repeat the treatment with the VivaStyle gel every day if possible or according to the schedule prescribed by your dentist.

Notes for patients:

- Overfilling the tray does not result in faster or better results. Instead, it may irritate the gums.
- Do not eat, drink or smoke while wearing the tray.
- Until you have finished the full treatment: avoid consuming citrus fruits and fruit juices, as they can promote hypersensitivity.
- Avoid, or at least try to minimize, the consumption of coffee, tea, red wine and colas as well smoking to prevent the risk of staining the teeth.
- The gel only whitens natural teeth. After the treatment, the appearance of existing restorations in the front teeth may no longer match that of the whitened teeth. Therefore, these restorations may have to be replaced.
- The treatment with VivaStyle 16% should be completed at least two weeks prior to the placement of tooth-coloured restorations.
- The teeth cannot be made to look whiter than their natural colour.

If you have any questions about your treatment or if you experience uncomfortable sensitivity, please contact your dental office.

Warning

Avoid contact of the material with the skin, mucous membrane and eyes. After accidental contact, rinse the affected area with copious amounts of water. If contact with the eyes occurs, immediately rinse with plenty of water and seek medical advice. Keep material out of the reach of children.

Shelf life and storage

- Store at < 30 °C / < 86 °F 
- Protect VivaStyle syringes from heat and sunlight
- Date of expiration: see label/packages

The product has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the product for its suitability and use for any purpose not explicitly stated in the Instructions.

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