## INTENDED USE

DE Rapid Cold Pack intended for cryotherapy

## PRESCRIPTION

DE Rapid Cold Pack is indicated in case of bruises, hematomas, traumas and muscle injuries. It relieves the perception of pain and facilitates the reduction of edema. Recommended treatment: 20/30 minutes. Always follow the doctor's prescription.

## INSTRUCTION FOR USE

1. Take the bag from one the flap and shake it.
2. Squeeze the middle part of the bag with both hands until the bubble inside bursts.
3. Shake well the bag to mix the content.
4. Apply the bag on the affected part.
5. Occasionally shake the bag to maintain the cold for a prolonged period.

## ATTENTION

- Do not use if the bag is damaged and do not break the bag.
- In case of bag content contact with skin or eyes, rinse with plenty of water.
- The bag content stains and is harmful if swallowed.
- If the bag content is ingested, consult a doctor immediately. The content is noxious.
- For external use only, do not use on open wounds.
- Irritation or sensitization may be caused by individual intolerance to cellulose and nonwoven.
- In case of cardiovascular and circulatory diseases, diabetes and/or skin insensibility (Raynaud's phenomenon), use the device only after consulting the doctor. Consult the doctor before using on newborn children, aged people and pregnant women
- Keep out of children's reach.
- Do not freeze the unit.
- Urban waste, it shall be disposed of in accordance with current regulation. Do not disperse in the environment.
- Report any serious incident that has occurred to the manufacturer and to the competent authority.
- Absolute contraindications: Raynaud's phenomenon, cold allergy, cryoglobulinemia, and paroxysmal cold hemoglobinuria.
- Relative contraindications: a rheumatic condition, pheochromocytoma, anesthetic skin, or a patient who is unresponsive due to cardiovascular disease.

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