SURFACE DISINFECTION WIPES

Ready to use alcoholic wipes for disinfecting near patient and hygienically critical areas

- Alcohol based
- Quick and easy application
- Suitable for hygienically critical and near-patient areas

PRODUCT DESCRIPTION:

CyberTech Surface disinfection wipes are ready to use alcoholic wipes for disinfecting near patient and hygienically critical areas. It is not necessary to wipe the area again after using CyberTech Surface disinfection wipes as it does not leave any residue on surfaces.

AREAS OF APPLICATION:

Acc. to Medical Device Directive (MDD): quick disinfection and cleaning of alcohol-resistant, non-invasive medical devices.

Acc. to Biocidal Products Regulation (BPR): quick disinfection and cleaning of alcohol-resistant surfaces of all kinds.

SPECTRUM OF APPLICATION AND EXPOSURE TIME:

Surface disinfection (wipe disinfection), bactericide, yeasticidal as per VAH: 1 min.; tuberculocide (M. terrae): 1 min; limited virucide (including HBV, HIV, HCV), rotaviruses: 30 sec.; noroviruses (MNV): 1 min

APPLICATION:

Wipe surfaces with CyberTech Surface disinfection wipes until they are completely wet and adhere to the contact time. Details for filling and processing of the wipe dispenser systems are to be regarded. Suitable for all alcohol-resistant surfaces. Not suited for acrylic glass (PMMA).

COMPOSITION:

100 g ready to use solution contains: 45 g ethanol (CAS: 64-17-5).

SPECIAL INFORMATION:

Open the refill package. Place the contents, including bag, into the dispenser. Pull the first cloth from the middle of the roll into the lid of the dispenser and replace the lid on the container. Please refill the container only with the refill package of Disinfection Wipes. Warning: Flammable solid. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Keep

away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection. For Professional Use Only.

PRODUCT STATUS/LISTINGS:

Conform to the EU Biocidal Product Regulation (BPR, 528/2012). Conform to the EU guideline 93/42/EC for medical devices.

AVAILABLE DELIVERY FORMS

Itemnumber	Packaging
9882781	120 pcs refill pack
9882783	120 pcs container

Use disinfectants safely. Always read the label and product information before use.

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