SURFACE DISINFECTION WIPES , SENSITIVE

Ready to use wipes for rapid disinfection of alcohol resistant surfaces, especially sensitive surfaces

- **Streak free application**
- **Especially gentle on materials**
- **Effective against norovirus (MNV)**

PRODUCT DESCRIPTION:

CyberTech Surface disinfection wipes, sensitive are ready to use wipes for rapid disinfection of alcohol resistant surfaces, especially sensitive surfaces. CyberTech Surface disinfection wipes, sensitive are particularly suited for the cleaning and disinfecting of displays and keyboards as well as sensitive control- and touch panels. The combination of wipe and soaking solution is finely tuned to enable gentle and effective disinfection. CyberTech Surface disinfection wipes, sensitive come in a resealable user-friendly flowpack.

AREAS OF APPLICATION:

Acc. to the Medical Device Directive (MDD): quick disinfection and cleaning of alcoholresistant, non-invasive medical devices.

Acc. to the Biocidal Products Regulation (BPR): quick disinfection and cleaning of alcohol-resistant surfaces of all kinds.

SPECTRUM OF APPLICATION AND EXPOSURE TIME:

Surface disinfection (wipe disinfection), bactericide, yeasticidal as per VAH: 1 min.; tuberculocide (M. terrae): 1 min; limited virucide (including HBV, HIV, HCV), rotaviruses: 30 sec.; noroviruses (MNV): 1 min

APPLICATION:

Wipe surfaces with CyberTech Surface disinfection wipes, sensitive until they are completely wet and adhere to the contact time. CyberTech Surface disinfection wipes, sensitive are designated for the use with refillable wipe dispenser systems. Details for filling and processing of the wipe dispenser systems are to be regarded. Suitable for all alcohol-resistant surfaces. Not suited for acrylic glass (PMMA).

COMPOSITION:

100 g ready to use solution contains: 45 g ethanol (CAS: 64-17-5).

SPECIAL INFORMATION:

Open the refill package. Place the contents, including bag, into the dispenser. Pull the first cloth from the middle of the roll into the lid of the dispenser and replace the lid on the container. Please refill the container only with the refill package of Disinfection Wipes. Warning: Flammable solid. If medical advice is needed, have prod-

uct container or label at hand. Keep out of reach of children. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection. For Professional Use Only.

PRODUCT STATUS/LISTINGS:

Registered as a biocide product at [baua](#) (Federal Institute for Occupational Safety and Health) in line with the Biocide Act: BauA reg. No.:

AVAILABLE DELIVERY FORMS

Itemnumber	Packaging
9882784	120 pcs refill pack
9882786	120 pcs container

Use disinfectants safely. Always read the label and product information before use.

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