BUR DISINFECTION / ROTARY INSTRUMENT DISINFECTION

Ready-for-use combined preparation for disinfection and cleaning

- **Contains corrosion inhibitors**
- **For use in ultrasonic bath or bur containers**
- **High material compatibility**

PRODUCT DESCRIPTION:

CyberTech Bur disinfection is a ready to use solution for efficient disinfection of rotary instruments like burs and endodontic instruments. The solution has excellent bactericidal activity, contains corrosion inhibitors and has a pleasant smell.

AREAS OF APPLICATION:

Ready-to-use disinfection product for burs and rotary instruments. For use in an ultrasonic bath or rotary container.

SPECTRUM OF ACTION:

Bactericidal against MRSA, yeasticidal, virucidal against HBV, HIV, HCV and BVDV. Contact time: Bactericidal, high organic load - 5 mins) Effective against all enveloped viruses including HBV, HIV, HCV, influenza viruses - 5mins.

APPLICATION:

Place instruments into the solution immediately after use. All surfaces and openings which require disinfecting must be fully exposed to the solution. Depending on level of soiling, replenish solution frequently. After the period of disinfection, thoroughly rinse and dry instruments. Suitable for use in ultrasonic baths.

COMPOSITION:

100 g contains: 0.28 g N,N-didecyl-n-methyl-poly(oxyethyl) ammonium propionate; 0.19 g guanidine, N,N'-(1,3-propanediyl)bis-, N-coco alkyl derivatives, diacetates.

SPECIAL INFORMATION:

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

PRODUCT STATUS/LISTINGS:

Conform to the EU guideline 93/42/EC for medical devices.

AVAILABLE DELIVERY FORMS

Itemnumber	Packaging
9882810	2L bottle
9882811	5L canister

AVAILABLE ACCESSORIES

Itemnumber	Packaging
9796273	Dosing pump for 2L bottle
9796276	Tab/dosing device for 5L canister
9796278	Canister-key

Use of disinfectants safely. Always read the label and product information before use.

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